### Instructions For Use

REF: LPH 020-S / LPH 020 / LPH 020-20

## Del (20q) Deletion Probe

PROFESSIONAL USE ONLY

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Further information available at [www.cytocell.com](http://www.cytocell.com)

Fluorescence *In Situ* Hybridisation (FISH) is a technique that allows DNA sequences to be detected on metaphase chromosomes or in interphase nuclei from fixed cytogenetic samples. The technique uses DNA probes that hybridise to entire chromosomes or single unique sequences, and serves as a powerful adjunct to classic cytogenetics. Recent developments have meant that this valuable technique can now be applied as an essential diagnostic tool in prenatal, haematological and pathological chromosomal analysis. Target DNA, after fixation and denaturation, is available for annealing to a similarly denatured, fluorescently labelled DNA probe, which has a complementary sequence. Following hybridisation, unbound and non-specifically bound DNA probe is removed and the DNA is counterstained for visualisation. Fluorescence microscopy then allows the visualisation of the hybridised probe on the target material.

### Probe Information

Deletions of the long arm of chromosome 20 are a common chromosomal abnormality associated with myeloid malignancies, in particular myeloproliferative disorders (MPD), myelodysplastic syndromes (MDS) and acute myeloid leukaemia (AML)<sup>1</sup>. The deletions are predominantly interstitial<sup>2</sup>.

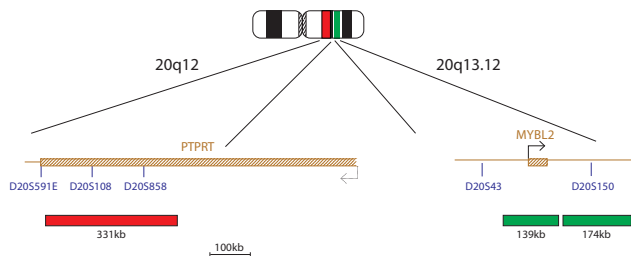
Often other cytogenetic abnormalities are present such as del(5q), -7/del(7q), +8, del(18q), +21 and rearrangements of 13q. Due to the relatively small size of the deletion and the lack of banding features on chromosome 20, FISH is particularly useful in detecting this abnormality.

The prognosis for MDS cases where del(20q) is the sole abnormality is good. However if secondary abnormalities are present, a poor outcome is indicated<sup>3</sup>. AML patients respond poorly to treatment and have reduced survival rates. The clinical outcome for MPD patients remains unchanged in the presence of the abnormality<sup>4</sup>.

The mechanism of leukaemogenesis associated with del(20q) is unknown, however, deletion of a tumour suppressor gene (TSG) is thought to cause the increased proliferation of the cancer cells<sup>4</sup>. Using RT-PCR analysis, Bench *et al* identified potential target genes in the region of overlap between the AML/MDS and MPD CDR at band 20q12. Five genes were expressed in both bone marrow and CD34 positive cells. Of the three previously identified genes, L(3)MBT regulates chromatin structure during mitosis, SFRS6 encodes a serine rich protein important to regulation of alternative splicing of mRNA, and MYBL2, a member of the MYB transcription factor family, is involved in cell cycle control<sup>5,6,7</sup>.

### Probe Specification

20q12, Red  
20q13.12, Green

The 20q12 probe, labelled in red, covers a 331kb region within the PTPRT gene and includes the D20S108 marker. The 20q13.12 probes, labelled in green (139kb and 174kb), cover the MYBL2 gene and includes the D20S150 marker.

### Materials Provided

**Probe:** 50µl per vial (5 tests), 100µl per vial (10 tests) or 200µl per vial (20 tests)

Amount of red 20q12 probe: 8-10ng/test

Amount of green 20q13.12 probe: 24-30ng/test

The probes are provided premixed in hybridisation solution (Formamide; Dextran Sulphate; SSC) and are ready to use.

**Counterstain:** 150µl per vial (15 tests) or 500µl per vial (50 tests)

The counterstain is DAPI antifade (ES: 0.125µg/ml DAPI (4,6-diamidino-2-phenylindole)).

### Warnings and Precautions

1. For *in vitro* diagnostic use. For professional use only.
2. Wear gloves when handling DNA probes and DAPI counterstain.
3. Probe mixtures contain formamide, which is a teratogen; do not breathe fumes or allow skin contact. Wear gloves, a lab coat, and handle in a fume hood. Upon disposal, flush with a large volume of water.
4. DAPI is a potential carcinogen. Handle with care; wear gloves and a lab coat. Upon disposal, flush with a large volume of water.
5. All hazardous materials should be disposed of according to your institution's guidelines for hazardous waste disposal.

### Storage and Handling

The Aquarius® kit should be stored at -20°C until the expiry date indicated on the kit label. The probe and counterstain vials must be stored in the dark.

### Equipment Necessary but not Supplied

1. Hotplate (with a solid plate and accurate temperature control up to 80°C).
2. Variable volume micropipettes and tips range 1µl - 200µl.
3. Water bath with accurate temperature control at 72°C.
4. Microcentrifuge tubes (0.5ml).
5. Fluorescence microscope (Please see Fluorescence Microscope Recommendation section).
6. Plastic or glass coplin jars.
7. Forceps.
8. Fluorescence grade microscope lens immersion oil.
9. Bench top centrifuge.
10. Microscope slides.
11. 24x24mm coverslips.
12. Timer.
13. 37°C incubator.
14. Rubber solution glue.

### Fluorescence Microscope Recommendation

For optimal visualisation of the probe we recommend a 100-watt mercury lamp and plan apochromat objectives x63 or x100. The Triple bandpass filter DAPI/FITC/Texas Red is optimal for viewing all fluorophores and DAPI simultaneously.

### Sample Preparation

The kit is designed for use on cultured peripheral blood cells or cultured bone marrow cells fixed in Carnoy's fixative that should be prepared according to the laboratory or institution guidelines. Prepare air dried samples on microscope slides according to standard cytogenetic procedures.

### FISH Protocol

(Note: Please ensure that exposure of the probe to laboratory lights is limited at all times).

### Slide preparation

1. Spot the cell sample onto a glass microscope slide. Allow to dry.
2. Immerse the slide in 2xSSC for 2 minutes at room temperature (RT) without agitation.
3. Dehydrate in an ethanol series (70%, 85% and 100%), each for 2 minutes at RT.
4. Allow to dry.

### Pre-Denaturation

5. Remove the probe from the freezer and allow it to warm to RT.
6. Ensure that the probe solution is uniformly mixed with a pipette.
7. Remove 10µl of probe per test, and transfer it to a microcentrifuge tube. Quickly return the remaining probe to -20°C.
8. Place the probe and the sample slide to prewarm on a 37°C (+/- 1°C) hotplate for 5 minutes.
9. Spot 10µl of probe mixture onto the cell sample and carefully apply a coverslip. Seal with rubber solution glue and allow the glue to dry completely.

### Denaturation

10. Denature the sample and probe simultaneously by heating the slide on a hotplate at 75°C (+/- 1°C) for 2 minutes.

### Hybridisation

11. Place the slide in a humid, lightproof container at 37°C (+/- 1°C) overnight.

### Post-Hybridisation Washes

12. Remove the coverslip and all traces of glue carefully.
13. Immerse the slide in 0.4xSSC (pH 7.0) at 72°C (+/- 1°C) for 2 minutes without agitation.
14. Drain the slide and immerse it in 2xSSC, 0.05% Tween-20 at RT (pH 7.0) for 30 seconds without agitation.
15. Drain the slide and apply 10µl of DAPI antifade onto each sample.
16. Cover with a coverslip, remove any bubbles and allow the colour to develop in the dark for 10 minutes.
17. View with a fluorescence microscope.

### Stability of Finished Slides

FISHed slides remain analysable for up to 1 month if stored in the dark at/or below RT.

## Procedural Recommendations

1. Baking or ageing of slides is not recommended as it may reduce signal fluorescence.
2. Hybridisation conditions may be adversely affected by the use of reagents other than those provided or recommended by CytoCELL Ltd.
3. The use of a calibrated thermometer is strongly recommended for measuring temperatures of solutions, waterbaths, and incubators as these temperatures are critical for optimum product performance.
4. The wash concentrations, pH and temperatures are important as low stringency can result in non-specific binding of the probe and too high stringency can result in a lack of signal.
5. Incomplete denaturation can result in lack of signal and over denaturation can also result in non-specific binding.

## Expected Results

In a normal cell there should be two red and two green signals (2R, 2G). Deleted cells may show one of the following signal patterns:

1. If a deletion is interstitial encompassing band q12 but not q13.12 and is hemizygous, one red and two green signals should be observed (1R, 2G).
2. If a deletion is interstitial encompassing band q12 but not q13.12 and is homozygous, no red and two green signals should be observed (0R, 2G).
3. If a deletion encompasses band q13.12 but not q12 and is hemizygous, two red and one green signals should be observed (2R, 1G).
4. If a deletion encompasses band q13.12 but not q12 and is homozygous, two red and no green signals should be observed (2R, 0G).
5. One red and one green signal pattern (1R, 1G) can be observed in case of monosomy 20 or hemizygous deletion of both bands on 20q.

## Limitations

Reporting and interpretation of FISH results should be consistent with professional standards of practice and should take into consideration other clinical and diagnostic information. This kit is intended as an adjunct to other diagnostic laboratory tests and therapeutic action should not be initiated on the basis of the FISH result alone.

## Additional Information

For additional product information please contact the CytoCELL Technical Support Department.

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### Patents and Trademarks
Aquarius and Cytocell are registered trademarks of Cytocell Ltd.
#### Cytocell Ltd.