Instructions For Use

REF: LPH 039-S / LPH 039 / LPH 039-20

CKS1B/CDKN2C (P18) Amplification/Deletion Probe

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Further information available at www.cytocell.com

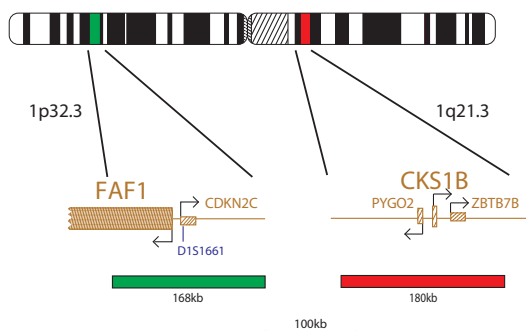
Fluorescence *In Situ* Hybridisation (FISH) is a technique that allows DNA sequences to be detected on metaphase chromosomes or in interphase nuclei from fixed cytogenetic samples. The technique uses DNA probes that hybridise to entire chromosomes or single unique sequences, and serves as a powerful adjunct to classic cytogenetics. Recent developments have meant that this valuable technique can now be applied as an essential diagnostic tool in prenatal, haematological and pathological chromosomal analysis. Target DNA, after fixation and denaturation, is available for annealing to a similarly denatured, fluorescently labelled DNA probe, which has a complementary sequence. Following hybridisation, unbound and non-specifically bound DNA probe is removed and the DNA is counterstained for visualisation. Fluorescence microscopy then allows the visualisation of the hybridised probe on the target material.

Probe Information

Structural abnormalities of chromosome 1 are frequently detected in multiple myeloma and have been correlated with more advanced disease¹. Amplification of 1q21 (CKS1B) is one of the most recurrent chromosomal aberrations in multiple myeloma³. Over-expression of the CKS1B gene up-regulates cell cycle progression, resulting in a more proliferative disease². This is related to the advanced phenotype of multiple myeloma and may therefore be associated with poor prognosis and disease progression³. Gain of 1q21 has been linked to inferior survival and further amplification is observed in disease relapse. It has been shown that gain of 1q21 copy number is not an independent prognostic factor in multiple myeloma and it is often associated with other chromosomal aberrations, most commonly t(4;14)(p16;q32) (IGH-FGFR3) and chromosome 13 deletion². The association of 1q21 gain and loss of chromosome 13 has been linked to the risk of conversion to overt disease². In a case study of multiple myeloma patients, it was discovered that 30% of chromosomal abnormalities mapped to chromosome 1, that most up-regulated genes mapped to chromosome 1q and down-regulated genes to chromosome 1p⁴. Gains of the long arm 1q are one of the most common genetic abnormalities in multiple myeloma⁵ and duplications of chromosome band 1q are frequently associated with disease progression^{3,6,7}. CDKN2C (P18), in band 1p32.3, is a tumour suppressor gene responsible for inducing apoptotic cell death and DNA fragmentation⁸. It is up-regulated by the expression of the cytokine IL-6 in multiple myeloma and homozygous deletion of the gene is associated with a more proliferative disease⁹. Although CDKN2C deletions have been reported to be rare in human malignancy, cytogenetic analyses have shown that abnormalities of 1p32-36 occur in around 16% of human multiple myeloma⁹.

Probe Specification

CKS1B, 1q21.3, Red
CDKN2C (P18), 1p32.3, Green

The CKS1B/CDKN2C product consists of a 180kb probe, labelled in red, covering the entire CKS1B gene and flanking regions, including the PYGO2

and ZBTB7B genes, and a green probe covering a 168kb region, including the entire CDKN2C gene, the D1S1661 marker and the centromeric end of the FAF1 gene.

Materials Provided

Probe: 50µl per vial (5 tests), 100µl per vial (10 tests) or 200µl per vial (20 tests)

Amount of red CKS1B probe: 12-15ng/test

Amount of green CDKN2C probe: 36-45ng/test

The probes are provided premixed in hybridisation solution (Formamide; Dextran Sulphate; SSC) and are ready to use.

Counterstain: 150µl per vial (15 tests) or 500µl per vial (50 tests)

The counterstain is DAPI antifade (ES: 0.125µg/ml DAPI (4,6-diamidino-2-phenylindole)).

Warnings and Precautions

1. For *in vitro* diagnostic use. For professional use only.
2. Wear gloves when handling DNA probes and DAPI counterstain.
3. Probe mixtures contain formamide, which is a teratogen; do not breathe fumes or allow skin contact. Wear gloves, a lab coat, and handle in a fume hood. Upon disposal, flush with a large volume of water.
4. DAPI is a potential carcinogen. Handle with care; wear gloves and a lab coat. Upon disposal, flush with a large volume of water.
5. All hazardous materials should be disposed of according to your institution's guidelines for hazardous waste disposal.

Storage and Handling

The Aquarius® kit should be stored at -20°C until the expiry date indicated on the kit label. The probe and counterstain vials must be stored in the dark.

Equipment Necessary but not Supplied

1. Hotplate (with a solid plate and accurate temperature control up to 80°C).
2. Variable volume micropipettes and tips range 1µl - 200µl.
3. Water bath with accurate temperature control at 72°C.
4. Microcentrifuge tubes (0.5ml).
5. Fluorescence microscope (Please see Fluorescence Microscope Recommendation section).
6. Plastic or glass coplin jars.
7. Forceps.
8. Fluorescence grade microscope lens immersion oil.
9. Bench top centrifuge.
10. Microscope slides.
11. 24x24mm coverslips.
12. Timer.
13. 37°C incubator.
14. Rubber solution glue.

Fluorescence Microscope Recommendation

For optimal visualisation of the probe we recommend a 100-watt mercury lamp and plan apochromat objectives x63 or x100. The Triple bandpass filter DAPI/FITC/Texas Red is optimal for viewing all fluorophores and DAPI simultaneously.

Sample Preparation

The kit is designed for use on cultured peripheral blood cells or cultured bone marrow cells fixed in Carnoy's fixative, that should be prepared according to the laboratory or institution guidelines. Prepare air dried samples on microscope slides according to standard cytogenetic procedures.

FISH Protocol

(Note: Please ensure that exposure of the probe to laboratory lights is limited at all times).

Slide preparation

1. Spot the cell sample onto a glass microscope slide. Allow to dry.
2. Immerse the slide in 2xSSC for 2 minutes at room temperature (RT) without agitation.
3. Dehydrate in an ethanol series (70%, 85% and 100%), each for 2 minutes at RT.
4. Allow to dry.

Pre-Denaturation

5. Remove the probe from the freezer and allow it to warm to RT.
6. Ensure that the probe solution is uniformly mixed with a pipette.
7. Remove 10µl of probe per test, and transfer it to a microcentrifuge tube. Quickly return the remaining probe to -20°C.
8. Place the probe and the sample slide to prewarm on a 37°C (+/- 1°C) hotplate for 5 minutes.
9. Spot 10µl of probe mixture onto the cell sample and carefully apply a coverslip. Seal with rubber solution glue and allow the glue to dry completely.

Denaturation

10. Denature the sample and probe simultaneously by heating the slide on a hotplate at 75°C (+/- 1°C) for 2 minutes.

Hybridisation

11. Place the slide in a humid, lightproof container at 37°C (+/- 1°C) overnight.

Post-Hybridisation Washes

12. Remove the coverslip and all traces of glue carefully.
13. Immerse the slide in 0.4xSSC (pH 7.0) at 72°C (+/- 1°C) for 2 minutes without agitation.
14. Drain the slide and immerse it in 2xSSC, 0.05% Tween-20 at RT (pH 7.0) for 30 seconds without agitation.
15. Drain the slide and apply 10µl of DAPI antifade onto each sample.
16. Cover with a coverslip, remove any bubbles and allow the colour to develop in the dark for 10 minutes.

- View with a fluorescence microscope.

Stability of Finished Slides

FISHed slides remain analysable for up to 1 month if stored in the dark at/or below RT.

Procedural Recommendations

- Baking or ageing of slides is not recommended as it may reduce signal fluorescence.
- Hybridisation conditions may be adversely affected by the use of reagents other than those provided or recommended by CytoCELL Ltd.
- The use of a calibrated thermometer is strongly recommended for measuring temperatures of solutions, waterbaths, and incubators as these temperatures are critical for optimum product performance.
- The wash concentrations, pH and temperatures are important as low stringency can result in non-specific binding of the probe and too high stringency can result in a lack of signal.
- Incomplete denaturation can result in lack of signal and over denaturation can also result in non-specific binding.

Expected Results

In a normal cell there should be two red and two green signals (2R, 2G). A cell with a deletion of 1p32.3 should have two red and one green signals (2R, 1G) and a cell with an amplification of 1q21 locus should have two green and multiple red signals.

Limitations

Reporting and interpretation of FISH results should be consistent with professional standards of practice and should take into consideration other clinical and diagnostic information. This kit is intended as an adjunct to other diagnostic laboratory tests and therapeutic action should not be initiated on the basis of the FISH result alone.

Additional Information

For additional product information please contact the CytoCELL Technical Support Department.
T: +44 (0)1223 294048
E: techsupport@cytoCELL.com
W: www.cytoCELL.com

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Patents and Trademarks
Aquarius and CytoCELL are registered trademarks of CytoCELL Ltd.
This product contains technology licensed from Life Technologies Corporation