Instructions For Use

REF: LPH 052-S / LPH 052 / LPH 052-20

P53/ATM Probe Combination

PROFESSIONAL USE ONLY

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Further information available at www.cytocell.com

Fluorescence *In Situ* Hybridisation (FISH) is a technique that allows DNA sequences to be detected on metaphase chromosomes or in interphase nuclei from fixed cytogenetic samples. The technique uses DNA probes that hybridise to entire chromosomes or single unique sequences, and serves as a powerful adjunct to classic cytogenetics. Recent developments have meant that this valuable technique can now be applied as an essential diagnostic tool in prenatal, haematological and pathological chromosomal analysis. Target DNA, after fixation and denaturation, is available for annealing to a similarly denatured, fluorescently labelled DNA probe, which has a complementary sequence. Following hybridisation, unbound and non-specifically bound DNA probe is removed and the DNA is counterstained for visualisation. Fluorescence microscopy then allows the visualisation of the hybridised probe on the target material.

Probe Information

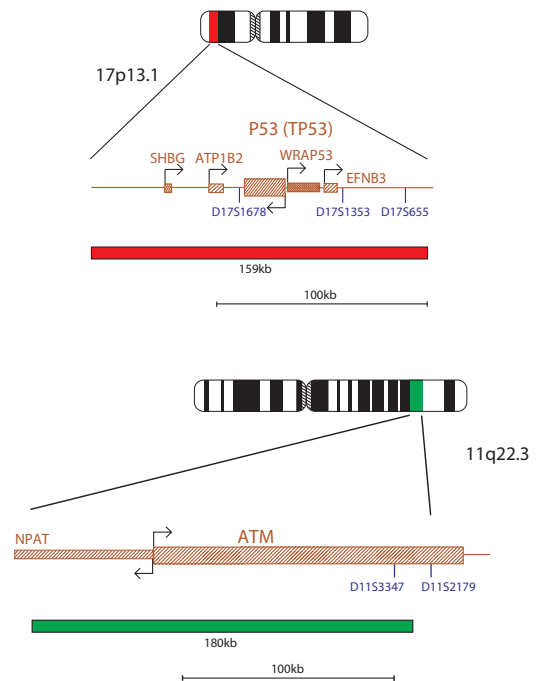
Although previously difficult to detect, the advent of FISH analysis of interphase cells from patients with B-CLL showed that around 17% of patients with the disease have deletions of the P53 (TP53) gene¹. As with ATM, deletions of P53 have important therapeutic implications for patients with B-CLL². Knowledge of the P53 deletion status in the patient should mediate the choice of therapy. P53 is a tumour suppressor gene and its protein product is responsible for the death of cells that contain damaged DNA. This is thought to be brought about by phosphorylation of P53 and the subsequent prevention of its repression by MDM2 (Mouse Double Minute 2 Homolog). This phosphorylation is mediated by ATM. In the absence of P53 activity, cells that cannot be repaired by ATM will continue to proliferate in their damaged state. Patients deleted for P53 may be rendered resistant to alkylating chemotherapeutic agents³ and purine analogues⁴ as these are designed to damage DNA in the cells that P53 would have destroyed. In the absence of P53, therefore, patients treated with these agents will harbour a proliferating population of damaged cells.

The protein kinase ATM (Ataxia-Telangiectasia Mutated) gene, located in 11q22.3, is frequently deleted in cases of B-CLL. ATM is an important checkpoint gene involved in the management of cell damage and its function is to assess the level of DNA damage in the cell and attempt repair by phosphorylating key substrates involved in the DNA damage response pathway. The ATM/P53 interaction in B-CLL has been shown to play an important role in the proliferation of lymphoid cancer⁵. It has been shown that ATM concurrently enhances the phosphorylation of P53⁶ should the damage be so great that the cell requires destruction by apoptosis (which is mediated by P53). Deletion of ATM removes this checkpoint activity and hence activation of P53. Thus, there is no attempt made to repair or to apoptose damaged cells, despite the presence of P53. In the absence of ATM, damaged cells are allowed to continue to proliferate.

Screening for deletions of ATM and/or P53 is vital to allow informed therapy choices for CLL patients, especially as deletions of P53 and ATM confer poor prognosis².

Probe Specification

P53, 17p13.1, Red
ATM, 11q22.3, Green

The P53 probe is 159kb, labelled in red and covers the whole P53 gene, extends 66kb telomeric to the gene and covers a region centromeric to the gene, to just beyond the marker D17S655. The ATM probe is 180kb, labelled in green, and covers the telomeric end of NPAT gene and the centromeric end of the ATM gene to just beyond the D11S3347 marker.

Materials Provided

Probe: 50µl per vial (5 tests), 100µl per vial (10 tests) or 200µl per vial (20 tests)

Amount of red P53 probe: 30-37.5ng/test

Amount of green ATM probe: 180-225ng/test

The probes are provided premixed in hybridisation solution (Formamide; Dextran Sulphate; SSC) and are ready to use.

Counterstain: 150µl per vial (15 tests) or 500µl per vial (50 tests)

The counterstain is DAPI antifade (ES: 0.125µg/ml DAPI (4,6-diamidino-2-phenylindole)).

Warnings and Precautions

1. For *in vitro* diagnostic use. For professional use only.
2. Wear gloves when handling DNA probes and DAPI counterstain.
3. Probe mixtures contain formamide, which is a teratogen; do not breathe fumes or allow skin contact. Wear gloves, a lab coat, and handle in a fume hood. Upon disposal, flush with a large volume of water.
4. DAPI is a potential carcinogen. Handle with care; wear gloves and a lab coat. Upon disposal, flush with a large volume of water.
5. All hazardous materials should be disposed of according to your institution's guidelines for hazardous waste disposal.

Storage and Handling

The Aquarius® kit should be stored at -20°C until the expiry date indicated on the kit label. The probe and counterstain vials must be stored in the dark.

Equipment Necessary but not Supplied

1. Hotplate (with a solid plate and accurate temperature control up to 80°C).
2. Variable volume micropipettes and tips range 1µl - 200µl.
3. Water bath with accurate temperature control at 72°C.
4. Microcentrifuge tubes (0.5ml).
5. Fluorescence microscope (Please see Fluorescence Microscope Recommendation section).
6. Plastic or glass coplin jars.
7. Forceps.
8. Fluorescence grade microscope lens immersion oil.
9. Bench top centrifuge.
10. Microscope slides.
11. 24x24mm coverslips.
12. Timer.
13. 37°C incubator.
14. Rubber solution glue.

Fluorescence Microscope Recommendation

For optimal visualisation of the probe we recommend a 100-watt mercury lamp and plan apochromat objectives x63 or x100. The Triple bandpass filter DAPI/FITC/Texas Red is optimal for viewing all fluorophores and DAPI simultaneously.

Sample Preparation

The kit is designed for use on cultured peripheral blood cells or cultured bone marrow cells fixed in Carnoy's fixative, that should be prepared according to the laboratory or institution guidelines. Prepare air dried samples on microscope slides according to standard cytogenetic procedures.

FISH Protocol

(Note: Please ensure that exposure of the probe to laboratory lights is limited at all times).

Slide preparation

1. Spot the cell sample onto a glass microscope slide. Allow to dry.
2. Immerse the slide in 2xSSC for 2 minutes at room temperature (RT) without agitation.
3. Dehydrate in an ethanol series (70%, 85% and 100%), each for 2 minutes at RT.
4. Allow to dry.

Pre-Denaturation

5. Remove the probe from the freezer and allow it to warm to RT.
6. Ensure that the probe solution is uniformly mixed with a pipette.
7. Remove 10µl of probe per test, and transfer it to a microcentrifuge tube. Quickly return the remaining probe to -20°C.
8. Place the probe and the sample slide to prewarm on a 37°C (+/- 1°C) hotplate for 5 minutes.
9. Spot 10µl of probe mixture onto the cell sample and carefully apply a coverslip. Seal with rubber solution glue and allow the glue to dry completely.

Denaturation

10. Denature the sample and probe simultaneously by heating the slide on a hotplate at 75°C (+/- 1°C) for 2 minutes.

Hybridisation

11. Place the slide in a humid, lightproof container at 37°C (+/- 1°C) overnight.

Post-Hybridisation Washes

12. Remove the coverslip and all traces of glue carefully.
13. Immerse the slide in 0.4xSSC (pH 7.0) at 72°C (+/- 1°C) for 2 minutes without agitation.
14. Drain the slide and immerse it in 2xSSC, 0.05% Tween-20 at RT (pH 7.0) for 30 seconds without agitation.
15. Drain the slide and apply 10µl of DAPI antifade onto each sample.
16. Cover with a coverslip, remove any bubbles and allow the colour to develop in the dark for 10 minutes.
17. View with a fluorescence microscope.

Stability of Finished Slides

FISHed slides remain analysable for up to 1 month if stored in the dark at/or below RT.

Procedural Recommendations

1. Baking or ageing of slides is not recommended as it may reduce signal fluorescence.
2. Hybridisation conditions may be adversely affected by the use of reagents other than those provided or recommended by Cytocell Ltd.
3. The use of a calibrated thermometer is strongly recommended for measuring temperatures of solutions, waterbaths, and incubators as these temperatures are critical for optimum product performance.
4. The wash concentrations, pH and temperatures are important as low stringency can result in non-specific binding of the probe and too high stringency can result in a lack of signal.
5. Incomplete denaturation can result in lack of signal and over denaturation can also result in non-specific binding.

Expected Results

In a normal cell there should be two red and two green signals (2R, 2G). In a cell with an ATM deletion there should be two red and one green signals (2R, 1G) whilst a cell with a P53 deletion should have one red and two green signals (1R, 2G).

Limitations

Reporting and interpretation of FISH results should be consistent with professional standards of practice and should take into consideration other clinical and diagnostic information. This kit is intended as an adjunct to other diagnostic laboratory tests and therapeutic action should not be initiated on the basis of the FISH result alone.

Additional Information

For additional product information please contact the Cytocell Technical Support Department.
T: +44 (0)1223 294048
E: techsupport@cytocell.com
W: www.cytocell.com

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Patents and Trademarks
Aquarius and CytoceLL are registered trademarks of CytoceLL Ltd. This product contains technology licensed from Life Technologies Corporation and is available for human diagnostics or life science research use only.
CytoceLL Ltd.