Instructions For Use

REF: LPP xxR/G

Painting Probes

PROFESSIONAL USE ONLY

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Further information available at www.cytocell.com

Fluorescence *In Situ* Hybridisation (FISH) is a technique that allows DNA sequences to be detected on metaphase chromosomes or in interphase nuclei from fixed cytogenetic samples. The technique uses DNA probes that hybridise to entire chromosomes or single unique sequences, and serves as a powerful adjunct to classic cytogenetics. Recent developments have meant that this valuable technique can now be applied as an essential diagnostic tool in prenatal, haematological and pathological chromosomal analysis. Target DNA, after fixation and denaturation, is available for annealing to a similarly denatured, fluorescently labelled DNA probe, which has a complementary sequence. Following hybridisation, unbound and non-specifically bound DNA probe is removed and the DNA is counterstained for visualisation. Fluorescence microscopy then allows the visualisation of the hybridised probe on the target material.

Probe Information

Whole chromosome painting probes are specific for each chromosome and are derived from flow-sorted chromosomes. They are libraries of many sequential sequences extending along the whole length of the chromosome.

Probe Specification

The probes are directly labelled with either a red or a green fluorophore. For detailed probe specifications refer to Table 1.

Table 1: Probe Specifications

Chromosome	Catalogue Number*
1	LPP 01R/G
2	LPP 02R/G
3	LPP 03R/G
4	LPP 04R/G
5	LPP 05R/G
6	LPP 06R/G
7	LPP 07R/G
8	LPP 08R/G
9	LPP 09R/G
10	LPP 10R/G
11	LPP 11R/G
12	LPP 12R/G
13	LPP 13R/G
14	LPP 14R/G
15	LPP 15R/G
16	LPP 16R/G
17	LPP 17R/G
18	LPP 18R/G
19	LPP 19R/G
20	LPP 20R/G
21	LPP 21R/G
22	LPP 22R/G
X	LPP 0XR/G
Y	LPP 0YR/G

*R specifies a red label and G specifies a green label

This Aquarius® kit contains only one of the probes from the range of directly labelled whole chromosome painting probes.

Materials Provided

Probe: 50µl per vial (5 tests)

Amount of red whole chromosome painting probe: 18-38ng/test

Amount of green whole chromosome painting probe: 56-94ng/test

The probes are provided in hybridisation solution (Formamide; Dextran Sulphate; SSC) and are ready to use. They are labelled with either a red or a green fluorophore.

Counterstain: 150µl per vial (15 tests)

The counterstain is DAPI antifade (ES: 0.125µg/ml DAPI (4,6-diamidino-2-phenylindole)).

Warnings and Precautions

1. For *in vitro* diagnostic use. For professional use only.
2. Wear gloves when handling DNA probes and DAPI counterstain.
3. Probe mixtures contain formamide, which is a teratogen; do not breathe fumes or allow skin contact. Wear gloves, a lab coat, and handle in a fume hood. Upon disposal, flush with a large volume of water.
4. DAPI is a potential carcinogen. Handle with care; wear gloves and a lab coat. Upon disposal, flush with a large volume of water.
5. All hazardous materials should be disposed of according to your institution's guidelines for hazardous waste disposal.

Storage and Handling

The Aquarius® kit should be stored at -20°C until the expiry date indicated on the kit label. The probe and counterstain vials must be stored in the dark.

Equipment Necessary but not Supplied

1. Hotplate (with a solid plate and accurate temperature control up to 80°C).
2. Variable volume micropipettes and tips range 1µl - 200µl.
3. Water bath with accurate temperature control at 72°C.
4. Microcentrifuge tubes (0.5ml).
5. Fluorescence microscope (Please see Fluorescence Microscope Recommendation section).
6. Plastic or glass coplin jars.
7. Forceps.
8. Fluorescence grade microscope lens immersion oil.
9. Bench top centrifuge.
10. Microscope slides.
11. 24x24mm coverslips.
12. Timer.
13. 37°C incubator.
14. Rubber solution glue.

Fluorescence Microscope Recommendation

For optimal visualisation of the probe we recommend a 100-watt mercury lamp and plan apochromat objectives x63 or x100. The Triple bandpass filter DAPI/FITC/Texas Red is optimal for viewing all fluorophores and DAPI simultaneously.

Sample Preparation

The kit is designed for use on cultured peripheral blood cells fixed in Carnoy's fixative that should be prepared according to the laboratory or institution guidelines. Prepare air dried samples on microscope slides according to standard cytogenetic procedures.

FISH Protocol

(Note: Please ensure that exposure of the probe to laboratory lights is limited at all times).

Slide preparation

1. Spot the cell sample onto a glass microscope slide. Allow to dry.
2. Immerse the slide in 2xSSC for 2 minutes at room temperature (RT) without agitation.
3. Dehydrate in an ethanol series (70%, 85% and 100%), each for 2 minutes at RT.
4. Allow to dry.

Pre-Denaturation

5. Remove the probe from the freezer and allow it to warm to RT.
6. Ensure that the probe solution is uniformly mixed with a pipette.
7. Remove 10µl of probe per test, and transfer it to a microcentrifuge tube. Quickly return the remaining probe to -20°C.
8. Place the probe and the sample slide to prewarm on a 37°C (+/- 1°C) hotplate for 5 minutes.
9. Spot 10µl of probe mixture onto the cell sample and carefully apply a coverslip. Seal with rubber solution glue and allow the glue to dry completely.

Denaturation

10. Denature the sample and probe simultaneously by heating the slide on a hotplate at 75°C (+/- 1°C) for 2 minutes.

Hybridisation

11. Place the slide in a humid, lightproof container at 37°C (+/- 1°C) overnight.

Post-Hybridisation Washes

12. Remove the coverslip and all traces of glue carefully.
13. Immerse the slide in 0.4xSSC (pH 7.0) at 72°C (+/- 1°C) for 2 minutes without agitation.
14. Drain the slide and immerse it in 2xSSC, 0.05% Tween-20 at RT (pH 7.0) for 30 seconds without agitation.
15. Drain the slide and apply 10µl of DAPI antifade onto each sample.
16. Cover with a coverslip, remove any bubbles and allow the colour to develop in the dark for 10 minutes.
17. View with a fluorescence microscope.

Suggestions for mixing probes

1. Mixing one red probe and one green probe: 5µl of red probe and 5µl of green probe per test.
2. Mixing one red, one green and a yellow probe: 3µl of red probe, 3µl of green probe and for the yellow: 2µl of red probe and 2µl of green probe for the same chromosome per test.

These volumes are for use as recommendations only. These volumes may require optimisation.

Stability of Finished Slides

FISHed slides remain analysable for up to 1 month if stored in the dark at/or below RT.

Procedural Recommendations

1. Baking or ageing of slides is not recommended as it may reduce signal fluorescence.
2. Hybridisation conditions may be adversely affected by the use of reagents other than those provided or recommended by CytoCELL Ltd.
3. The use of a calibrated thermometer is strongly recommended for measuring temperatures of solutions, waterbaths, and incubators as these temperatures are critical for optimum product performance.
4. The wash concentrations, pH and temperatures are important as low stringency can result in non-specific binding of the probe and too high stringency can result in a lack of signal.
5. Incomplete denaturation can result in lack of signal and over denaturation can also result in non-specific binding.

Expected Results

1. The acrocentric chromosome painting probes contain short arm material. This is shared between the D & G group chromosomes, so cross-hybridisation in these regions may be observed.
2. The chromosome 16 whole chromosome painting probe may show faint cross-hybridisation to the heterochromatic regions of chromosome Y.
3. The Y chromosome painting probe contains the pseudoautosomal regions common with the X chromosome. Consequently, cross-hybridisation may be observed in these regions.
4. The X chromosome painting probe contains the pseudoautosomal regions common with the Y chromosome. Consequently, cross-hybridisation may be observed in these regions.
5. Chromosomes 1, 5 and 19 have centromeric DNA sequences in common. Consequently cross-hybridisations in these regions may be observed between these chromosomes.
6. Chromosome 1 whole chromosome painting probe may show a brighter signal at 1p36.
7. Whole chromosome painting probes do not cover the heterochromatic blocks in chromosomes 1, 9 and 16.
8. The chromosome 21 whole chromosome painting probe may show cross-hybridisation to the subtelomere region of chromosome 1 p-arm.

Limitations

Reporting and interpretation of FISH results should be consistent with professional standards of practice and should take into consideration other clinical and diagnostic information. This kit is intended as an adjunct to other diagnostic laboratory tests and therapeutic action should not be initiated on the basis of the FISH result alone.

Additional Information

For additional product information please contact the CytoCELL Technical Support Department.
T: +44 (0)1223 294048
E: techsupport@cytoCELL.com
W: www.cytoCELL.com

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Patents and Trademarks
Aquarius and Cytocell are registered trademarks of Cytocell Ltd.
Cytocell Ltd.